


THE ANAPEN® TRAINING DEVICE

The Anapen® Training Device (Trainer) is a dummy device which allows the user to become familiar with the device and how it works. **The Trainer does not have a needle and does not contain adrenaline.** Anapen® Trainers can be used repeatedly but will tire with time and excessive use. Follow the instructions for firing and resetting to maximise the life of the device.

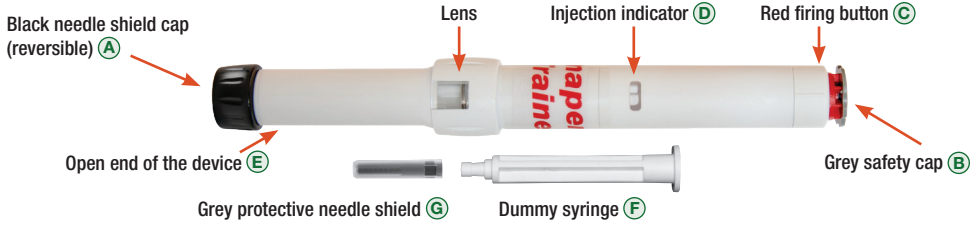

FIRING THE ANAPEN® TRAINER

- Remove the black needle shield cap **A** by pulling hard, in the direction of the arrow. This also removes a grey protective needle shield **G**.
- Remove the grey safety cap **B** from the red firing button **C**, by pulling as indicated by the arrow.
- Hold the open end of the device **E** against the outer part of your thigh.
- Once the open end of the device **E** is against the outer part of your thigh, press the red firing button **C** so that it clicks. Keep holding the Anapen Auto-Injector against your thigh for 10 seconds. Slowly remove Anapen from your thigh.
- The injection indicator **D** will have turned red. This shows that the injection is complete. (If the injection indicator is not red, you must repeat the injection with a new Anapen.)
- After the injection, click the wide end of the black needle shield cap **A** back on the open end of the device (as indicated by the arrow).

HOW TO RESET THE ANAPEN® TRAINER

- Replace the grey protective needle shield **G** on dummy syringe **F**.
! Before moving to the next step, place the grey protective needle shield back onto the end of the plastic dummy syringe.
 - Fit the open end of the device **E** into the thin end of the black needle shield cap **A**.
 - Push the black cap **A** until it clicks with the help of a hard surface.
 - Replace the grey safety cap **B** on the red firing button **C**.
! Stand the training pen upright with the black boot remover against a hard surface, and reattach the grey safety cap.
- Anapen® TRAINER is now ready for practice use again.**

Distributed by Allergy Concepts Pty Ltd
Level 1, 83 York Street Sydney NSW 2000
Email: info@anapen.com.au
www.anapen.com.au
JAN 2022